

***Patient transfer  
made easy...***



**PATSLIDE**®  
*Patient Transfer*

# With the Patslide® Patient Transfer, patients can be transferred in 3 easy steps



1. The drawsheet beneath the recumbent patient is first loosened on both sides and the receiving trolley is butted against the side of the bed. The person transferring the patient reaches across and grasps the drawsheet at the patient's shoulder and pelvic level and slightly tilts the patient towards him/herself. The receiving person slips the edge of the **PATSLIDE Patient Transfer** beneath the patient who is then lowered.



2. The **PATSLIDE Patient Transfer** is beneath the drawsheet and is resting partially on both the trolley and the bed, creating a smooth bridge. The receiving person grasps the drawsheet at the thoracic and mid-femoral levels and draws the patient towards him/herself as the transferring person assists with a gentle lift. The patient slides comfortably and easily to the centre of the trolley.



3. The **PATSLIDE Patient Transfer** is now gripped by the carry handle and withdrawn. The transfer has been completed and with a minimum of patient discomfort.

**PATSLIDE®** is not a lifting device and should not be used as such. The use of **PATSLIDE®** in any way other than as described is neither recommended nor approved.

When you consider the number of patient transfers made daily and the effort involved, a **PATSLIDE®** becomes an essential part of hospital equipment in Wards, Casualty, Operating Theatres, Radiology, in fact, anywhere that speed and ease of operation is needed.

**PATSLIDE®** not only makes the job easier and safer, but also reduces the risk of injury to nursing staff when transferring patients. The distress and discomfort that patients can experience when being transferred is eliminated.

**PATSLIDE®** has proven itself through daily use in hospitals around the world. This unique product has reduced the incidence of back strain occurring during patient transfer; a factor which previously contributed to many lost working hours and long-term staff disability.

**PATSLIDE®** is available in the following sizes:

	LENGTH	WIDTH
ALL PURPOSE	1525 mm	635 mm
THEATRE	1525 mm	560mm
SLIMLINE	1400 mm	480 mm
DUO-SET (2PIECE)	750 mm	635 mm each

**PATSLIDE®** is:

- Safe and easy to use
- Static Insulative
- Lightweight and Resilient
- Radio-Lucent
- Highly Impact Tolerant
- Easy to clean
- Maintenance Free

# PATSLIDE

Patient Transfer

Manufactured in Australia by: Briggate Medical Company  
A Division of Drewson International Pty Ltd A.B.N. 92 006 609 559  
23 - 25 Lakewood Blvd Braeside, Victoria 3195, Australia  
Telephone: +61 3 9580 5377 Facsimile: +61 3 9580 3688



Briggate Medical Company's Authorised Representative in the European Community (as defined in Article 14 (2) of the Council Directive: 93/42/EEC) is: Balwood Enterprises Ltd, 30 Vicarage Close, Langford, Biggleswade, Bedfordshire SG18 9QS England.

Distributed by:

AXIM -

Tel: 011 314 0140

FAX: 011 314 0141

Helmien van Staden 082 928 5516